



## REUSABLE TEXTILE MASKS

We are a **Romanian** company  
producing and distributing textiles,  
with over **20 years of experience** in apparel industry.

In these difficult times,  
we have chosen to invest all the resources  
of the company in the development  
of innovative products to support  
the current needs of the market.  
While adjusting to the present global situation,  
we have adapted all the capacity  
of our own production facilities  
towards the manufacturing  
**masks and other protection products.**

### OUR HYGIENIC MASKS ARE RECOMMENDED FOR:

- General population when they leave the house for shopping, professional interest, walking animals, physical activity
- Drivers and all public transport participants
- Students and teachers
- People working in unventilated or insufficiently vented places

### PURPOSE OF THE PRODUCT:

- Protecting the respiratory airways against atmospheric agents that could cause allergies, such as dust, pollen, pollution
- Role of protection of the respiratory tract against the saliva of the surrounding people and against other external fluids
- Role of protection of the working environment from the saliva drops of the semi-mask user, for activity in clean rooms, such as food industry, electron industry
- Role of protection of the face against allergies due to the use of antibacterial cotton material
- Role of maintaining hygiene in the mouth and nose against direct contact with the user's hands.





## HYGIENIC COTTON MASK FOR KIDS\*

DOUBLE LAYER

### REUSABLE MASK

**SIZES:** 3-6 years, 7 -11 years, 12-14 years  
20 pcs/ pack or individual package\*\*

#### *Composition:*

**Double layer of cotton with nose clips**

#### *Instructions for use:*

**Place the mask on the face by adjusting the two elastic strings behind the ears. Open the plies from the nose and chin, so that the mask takes the shape of the face. Using both hands, verify that the mask fits tightly on the face, covering the nose and chin. The mask can be reused after washing.**

#### *Care instructions:*

**Wash the mask at 40 degrees, let it air dry and then steam it, avoiding the elastic ear strings.**

**Maximum recommended time for use is 4 hours.**



\* Multiple color and print options available.  
\*\*Price is negotiable depending on quantity.  
Individual packaging and labelling is available for extra cost.





## HYGIENIC COTTON MASK

### REUSABLE MASK

20 pcs/ pack or individual package\*

#### *Composition*

**100% Cotton**


#### *Instructions for use:*

Place the mask on the face by adjusting the two elastic strings behind the ears. Open the plies from the nose and chin, so that the mask takes the shape of the face. Using both hands, verify that the mask fits tightly on the face, covering the nose and chin. The mask can be reused after washing.


#### *Care instructions:*

Wash the mask at 40 degrees, let it air dry and then steam it, avoiding the elastic ear strings.

Maximum recommended time for use is 4 hours.

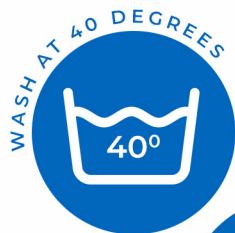

\*Price is negotiable depending on quantity.  
Individual packaging and labelling is available for extra cost.

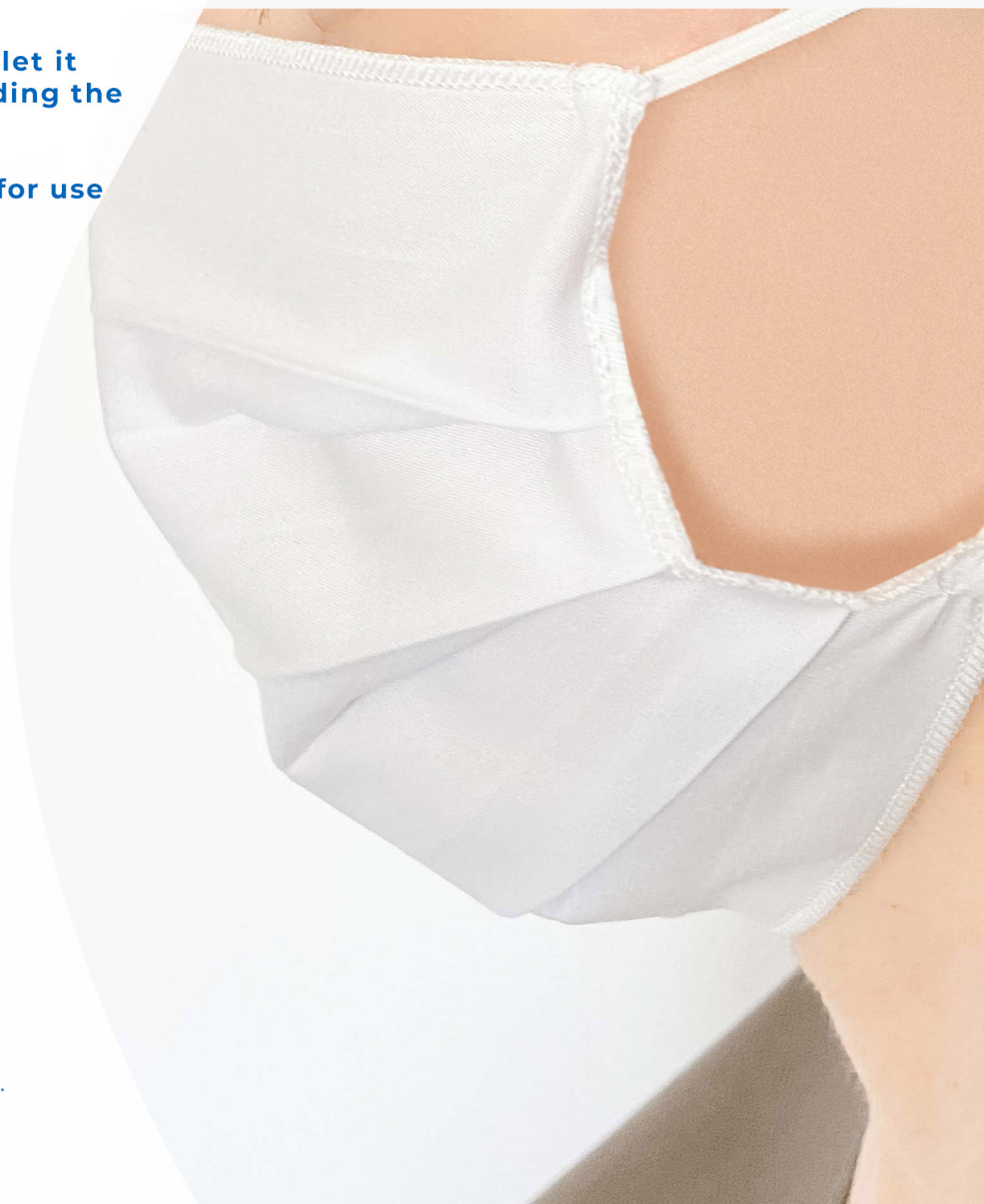





## PERSONALIZED HYGIENIC COTTON MASK\*

### REUSABLE MASK

20 pcs/ pack or individual package\*\*

*Composition:*

**100% Cotton**

*Instructions for use:*

Place the mask on the face by adjusting the two elastic strings behind the ears.  
Open the plies from the nose and chin, so that the mask takes the shape of the face.  
Using both hands, verify that the mask fits tightly on the face, covering the nose and chin.  
The mask can be reused after washing.

*Care instructions:*

Wash the mask at 40 degrees, let it air dry and then steam it, avoiding the elastic ear strings.

Maximum recommended time for use is 4 hours.



\* Price may differ depending on embroidered area.

\*\*Price is negotiable depending on quantity.  
Individual packaging and labelling is available for extra cost.





## HYGIENIC MASK COTTON + NON WOVEN

DOUBLE LAYER

### REUSABLE MASK

20 pcs/ pack or individual package\*

#### Composition:

**Inner layer: 100% Cotton**

**Outer layer : 100% Polypropylene**

#### Instructions for use:

**Place the mask on the face by adjusting the two elastic strings behind the ears.**

**Open the plies from the nose and chin, so that the mask takes the shape of the face.**

**Using both hands, verify that the mask fits tightly on the face, covering the nose and chin.**

**The mask can be reused after washing.**


#### Care instructions:

**Wash the mask at 40 degrees, let it air dry and then steam the cotton layer, avoiding the elastic ear strings.**

**Maximum recommended time for use is 4 hours.**



\*Price is negotiable depending on quantity  
Individual packaging and labelling is available for extra cost

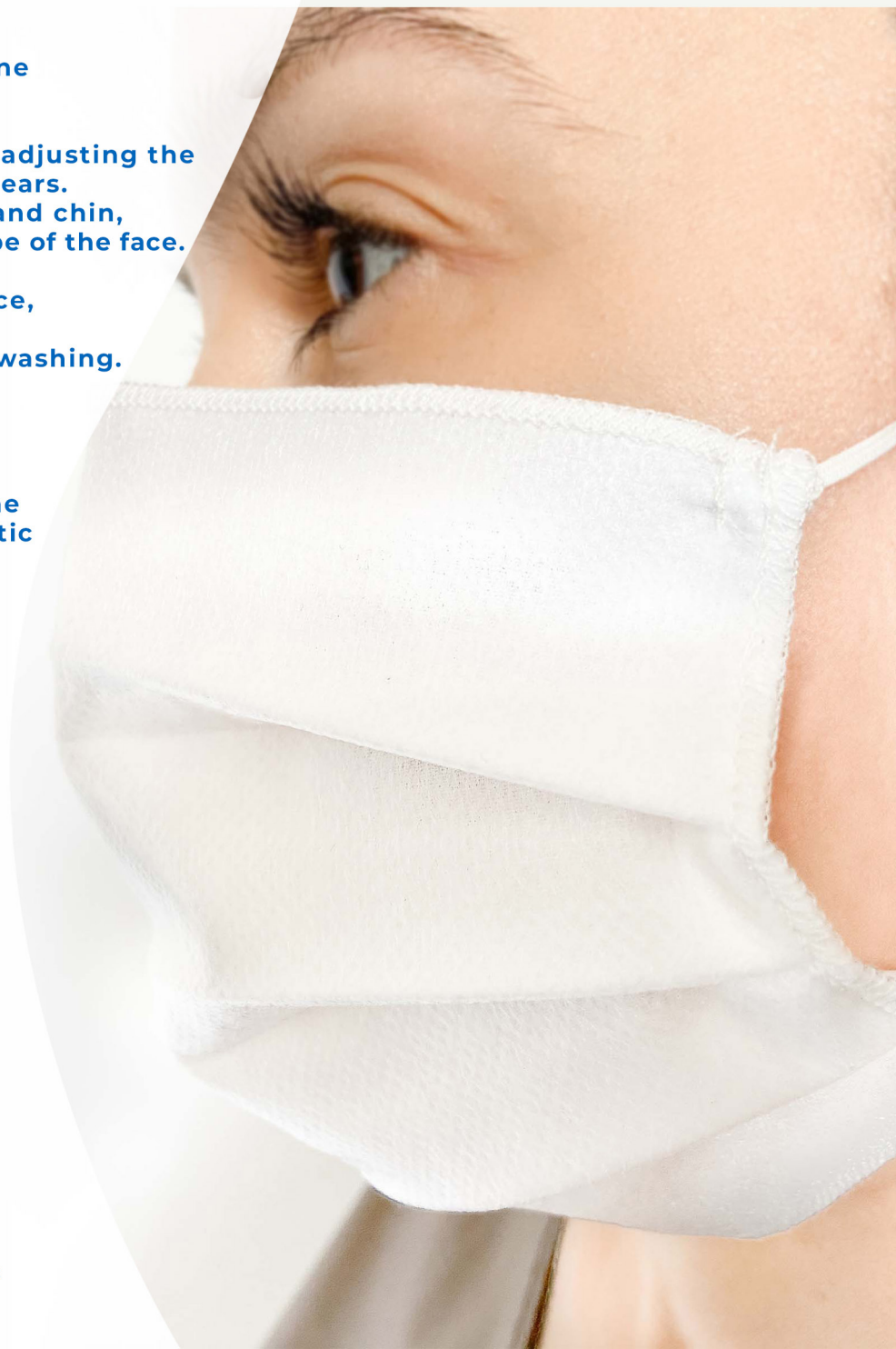





# HYGIENIC COTTON MASK WITH FILTER POCKET AND NOSE CLIPS

DOUBLE LAYER

## REUSABLE MASK

20 pcs/ pack or individual package\*

### Composition:

Inner layer: 100% Cotton, with side pocket for filter

Outer layer: 100% Cotton

Filter: waterproof microfibre (2pcs)

### Instructions for use:


Place the mask on the face, adjusting the two elastic strings behind the ears and the nose clips so the mask fits the face perfectly.

The mask can be reused after washing.  
The filter is for single use.

### Care instructions:

Wash the mask at 40 degrees, let it air dry and then steam it, avoiding the elastic ear strings.


Maximum recommended time for use is 4 hours.

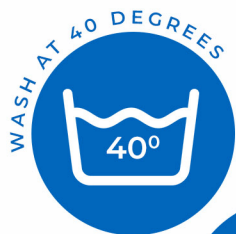

BACK VIEW



WATERPROOF MICROFIBRE

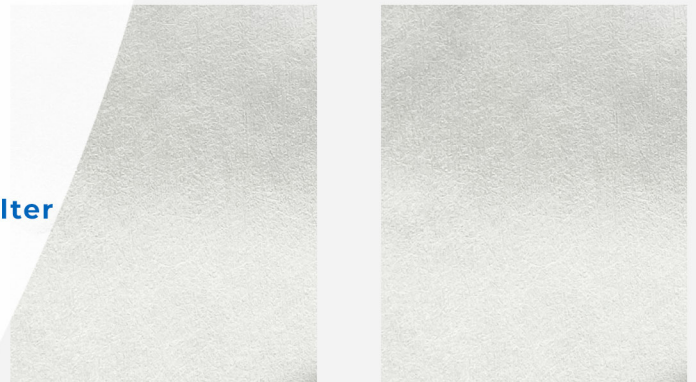

FRONT VIEW



SIDE VIEW



\*Price is negotiable depending on quantity  
Individual packaging and labelling is available for extra cost





## MASK DISPLAY



## INDIVIDUAL PACKAGING AND LABELING



### CONTACT:

Adriana Elena Poncu  
Commercial Manager  
Mobile: +40 736 363 438  
E-mail: aponcu@nissa.com

Lavinia Mihailescu  
Retail Manager  
Mobile: +40 737 796 004  
E-mail: lmihailescu@nissa.ro

Mihaela Bildea  
Export Manager  
Mobile: +40 731 790 270  
E-mail: mbildea@nissa.ro